


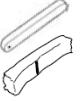






STEP 1: SOIL REMOVAL

- Before disinfection, disassemble and clean the breathing tube and chamber with a mild detergent solution.
- Rinse thoroughly under running potable tap water to remove detergent.
- Allow to thoroughly air dry before proceeding with high-level disinfection.

Failure to properly clean these items may result in inadequate disinfection.

STEP 2: HIGH-LEVEL DISINFECTION

| | | NOT RECOMMENDED FOR HIGH-LEVEL DISINFECTION | PASTEURIZATION | STERRAD® 100S | CIDEX® OPA |
|--|---|---|--|--|---|
| | | These parts are not recommended for high-level disinfection and should be replaced if the CPAP is used between multiple patients. | Soak the chamber in a hot-water bath for 30 minutes at 158°F (70°C). | Follow the manufacturer's instructions for the Sterrad® 100S sterilization system. | Immerse the chamber in Cidex® OPA for 12 minutes. Remove all parts thoroughly in clean water. |
|  | 900HC221 (SleepStyle™ 200 Series breathing tube) | ☑ | | | |
|  | HC385 (SleepStyle™ 200 Series standard chamber) | ☑ | | | |
|  | HC355 (SleepStyle™ 200 Series long-life chamber) | | ☑ | ☑ | ☑ Refer to the Operating Manual (185042488) for detailed instructions. |
|  | FILTER | ☑ | | | |
|  | HC365 (SleepStyle™ 600 Series standard chamber) | ☑ | | | |
|  | 900HC522 (SleepStyle™ 600 Series heated breathing tube) | | ☑ | | ☑ Refer to the Healthcare Provider Manual (185043061) for detailed instructions. |
|  | OUTLET PORT (SleepStyle™ 600 Series CPAP) | | | | ☑ Refer to the Outlet Cleaning Kit (185043557) for detailed instructions. |
|  | HC360 (SleepStyle™ 600 Series long-life chamber) | | ☑ | ☑ | ☑ Refer to the Operating Manual (185043383) for detailed instructions. |

ITEMS RECOMMENDED FOR HIGH-LEVEL DISINFECTION HAVE BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS.

Do not use the items if damaged. Note that it is the user's responsibility to qualify any deviations from the recommended methods. The information provided in this document is an indication only. Please refer to the User Instructions provided with each item for the most up-to-date information.