

Multi-Patient Cleaning Instructions for Fisher & Paykel Healthcare Interfaces










Quick Reference*

Step 1: Soil Removal

- Before disinfection, disassemble thoroughly, cleaning the mask with a soft bristled brush and a mild detergent solution.
- Rinse thoroughly under running potable tap water to remove detergent.
- Allow to thoroughly air dry before proceeding with high level disinfection.

Failure to properly clean this device may result in inadequate disinfection.

Step 2: High Level Disinfection

		Thermal Disinfection	Hexanios G+R®	CIDEX® OPA	Autoclave	Sterrad® 100S	Sporox® II
		Soak parts in hot water bath.	Follow manufacturers' instructions.	Follow manufacturers instructions for high level disinfection. Immerse at 20 °C (68 °F). Rinse thoroughly in potable water.	Gravity displacement type autoclave at 121 °C (250 °F). Enclose components in a sterilization peel pouch.	Dry thoroughly before disinfecting. Follow manufacturer's instructions for the Sterrad 100S Sterilization System.	Follow manufacturers instructions for high level disinfection. Immerse at 20 °C (68 °F). Rinse thoroughly in potable water.
	Zest™ Nasal Mask (excluding Foam Cushion)	✓ 30 minutes at 70 °C (158 °F)	✓ Soak for 30 minutes	✓ Soak for 30 minutes			
	FlexiFit™ 405/406/407 Nasal Mask (excluding Foam Cushion and Diffuser)	✓ 30 minutes at 70 °C (158 °F)		✓ Soak for 12 minutes	✓ 15 minute exposure	✓	✓ Soak for 6 hours
	Aclaim® 2 Nasal Mask (excluding Foam Cushion and Diffuser)	✓ 30 minutes at 70 °C (158 °F)		✓ Soak for 30 minutes	✓ 30 minute exposure. NB: Do not auto clave the FlexiTube	✓	✓ Soak for 6 hours
	Forma™ Full Face Mask (excluding Foam Cushion)	✓ 30 minutes at 70 °C (158 °F)	✓ Soak for 30 minutes	✓ Soak for 30 minutes			
	FlexiFit™ 432 Full Face Mask (excluding Foam Cushion)	✓ 30 minutes at 75 °C (167 °C)		✓ Soak for 60 minutes	✓ 30 minute exposure	✓	
	FlexiFit™ 431 Full Face Mask (excluding Foam Cushion)	✓ 30 minutes at 75 °C (167 °F)		✓ Soak for 3 hours	✓ 30 minute exposure	✓	
	Opus™360 Nasal Pillows Mask (excluding Contoured Mask Frame)	✓ 30 minutes at 75 °C (167 °F)		✓ Soak for 30 minutes		✓	
	Oracle™ Oral Mask	✓ 10 minutes at 80 °C (176 °F)		✓ Soak for 30 minutes			✓ Soak for 30 minutes
	Foam Cushion, Diffusers, and contoured Mask Frame	Cannot be disinfected and should be replaced between patients.					
	Headgear	Headgear can be cleaned by washing in warm soapy water and leaving to dry.					

- **All masks have been validated to withstand 20 cycles of the above parameters.**
- Do not use the mask if damaged. Note that it is the user's responsibility to qualify any deviations from the recommended methods.
- The information provided in this document is an indication only.

* Please refer to the User instructions provided with each mask for the most up-to-date information.